INSTRUCTIONS FOR USE (EN)

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DEVICE DESCRIPTION/INTENDED USE

Offset Cup Impactors are hand-held, re-usable surgical instruments which are designed to seat an acetabular shell into a patient's acetabulum in Total Hip Arthroplasty and Hip Resurfacing Arthroplasty. They are intended for use with a surgical hammer to seat the acetabular shell to the bone of the patient.

The Offset Cup Impactor may incorporate a fixed tip or a removable Offset Impactor Tip.

Anticipated useful life Offset Cup Impactor: 600 use cycles

WARNINGS

- Do not exceed 137°C.
- Do not use highly alkaline (pH>9) solutions.
- Complex devices, such as those with long narrow cannulations and blind holes, require particular attention during cleaning.

LIMITATIONS ON REPROCESSING

- Repeated processing has minimal effect on these instruments. End of life is determined by wear and damage due to intended use.

CARE AT THE POINT OF USE

- Remove excess soil with disposable, non-shedding wipes.

PREPARATION FOR DECONTAMINATION

AND CLEANING

- Reprocess instruments as soon as is reasonably practical following use.

- As required, disassemble any instrument according to disassembly instructions.
- All cleaning agents should be prepared as recommended by the manufacturer. (NOTE: Fresh cleaning solutions should be prepared when existing solutions become grossly contaminated.)

CLEANING: MANUAL

- Equipment: enzyme solution, detergent, brush, purified running water, single use towelettes.
- Method:
 1. Rinse excess soil from instruments.
 2. Soak in enzyme solution (prepared to manufacturer's instructions, 17.4°C to 18.6°C) for 16 minutes.
 3. Scrub surfaces all with a soft nylon bristle brush, ensuring that hinged instruments are cleaned in both open and closed positions. (NOTE: Ensure that the full depth of cannulations and holes is reached.)
 4. Rinse under clean running water (13.9°C) for at least 3 minutes. Clean ultrasonically for 16 minutes in detergent prepared in accordance to the manufacturer's instructions, at a temperature of 42°C to 42.6°C.
 5. Rinse with running purified water (20.2°C) for at least 3 minutes. Ensure that running water passes through cannulations and that blind holes are repeatedly filled and emptied.
 6. Dry with single use towelettes.

CLEANING: AUTOMATED

- Equipment: Washer/disinfector, enzyme solution, purified water, detergent, demineralized water
- Method:
 1. Rinse excess soil from instrument.

2. Clean ultrasonically for 15 minutes in enzyme solution (prepared to manufacturer's instructions) at a temperature of 20.2°C to 24.8°C.
3. Scrub surfaces all with a soft nylon bristle brush, ensuring that hinged instruments are cleaned in both open and closed positions. (NOTE: Ensure that the full depth of cannulations and holes is reached.)
4. Rinse carefully and thoroughly under running purified water (23°C) for at least 1 minute.
5. Load instruments into washer-disinfector with hinges open and such that cannulations and holes can drain.
6. Run cycle, minimum 10 minutes wash at 93°C with a cleaning product prepared to the manufacturer's specifications and rinse thoroughly with demineralized water.
7. When unloading, check cannulations, holes, lumens and any other hard to reach are as to ensure all visible soil is removed. If necessary, repeat cycle and/or clean manually.

DISINFECTION

- Instruments must be terminally sterilized prior to surgical use.

DRYING

- When drying is achieved as part of a washer disinfector cycle, do not exceed 120°C.

MAINTENANCE, INSPECTION AND TESTING

- Visually inspect for damage and wear. If the instrument is damaged and worn, it is considered at the end of its life and should be discarded.
- Check hinged instruments for smooth movement.
- When the UDI carrier(s) is no longer readable, the instrument is to be discarded.

PACKAGING

- Instruments may be loaded into dedicated instrument trays or sterilization trays
- Wrap in accordance with local procedures using standard wrapping techniques.

STERILIZATION

- All Viant reusable surgical devices must be sterilized prior to use.
- Use a validated, properly maintained and calibrated steam sterilizer.
- The following cycles have been validated to provide a sterility assurance level of 10^{-6} :

Cycle Type	Temperature (Minimum) °Celsius	Exposure Time (Minimum)	Dry Time (Minimum)
Pre-vacuum	132°C	4 minutes	30 minutes
	134°C	3 minutes	30 minutes
	134°C	18 minutes	30 minutes

*Fully loaded cases/trays may require longer dry time. The instructions provided above have been validated by the manufacturer of the medical device as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing, as actually performed, using equipment, materials, and personnel in the reprocessing facility, achieve the desired result. This normally requires validation and routine monitoring of the process.

STORAGE

- Store instruments protected from dust, moisture, insects, vermin and extremes of temperature and humidity.

ADDITIONAL INFORMATION

- Viant devices should only be used by qualified personnel fully trained in the use of the surgical instruments and the relevant surgical procedures.
- Do not modify Viant instruments in any way and handle with care at all times. Surface scratches can increase wear and the risk of corrosion.
- Manual surgical instruments have a limited life-span which is determined by wear or damage due to repeated intended use. When a surgical instrument reaches the end of its functional life, clean the instrument of any and all biomaterial/biohazards and safely discard the instrument in accordance with applicable laws and regulations.

Explanation of Symbols

	Manufacturer
	Authorized representative in Switzerland
	Authorized representative in the European Community
	Date of Manufacture
	Batch code
	Catalogue number
	Non-sterile
	Consult instructions for use
	Caution
	Quantity

NOTE: Please refer to the package labeling for applicable symbols.

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© 2019-22 Viant Medical, LLC and its affiliates. All rights reserved.
Viant Medical, LLC
MDSS GmbH
MDSS CH GmbH